# OPERATION MANUAL

Please read this OPERATION MANUAL carefully before use, and file for future reference.

Air Motor Set

Air Motor

**FX205 M4 / FX205 B2**  
**FX205m M4 / FX205m B2**

Contra Angle Handpiece

**FX22 / FX23 / FX25 / FX25m**


Straight Handpiece

**FX65 / FX65m**


MADE IN JAPAN


OM-M0137E 003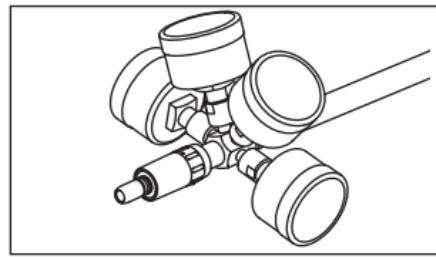


1

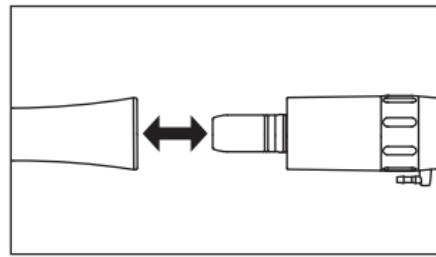
2

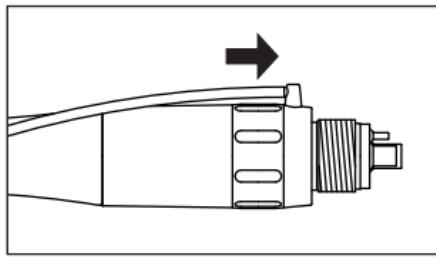
3

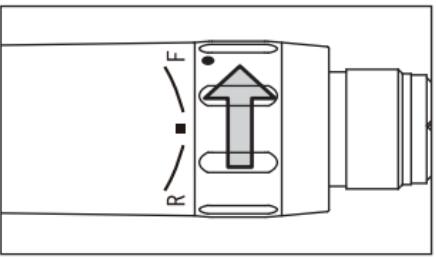
4

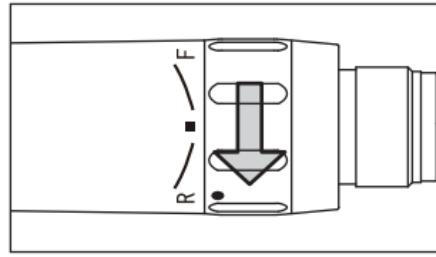
5

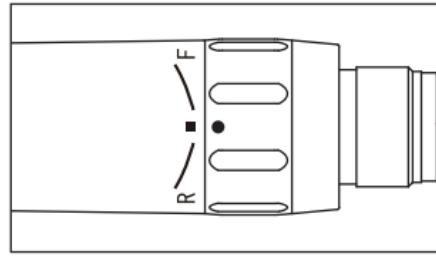
6

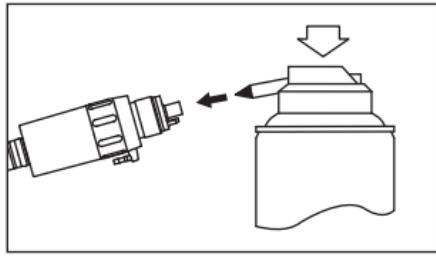
7

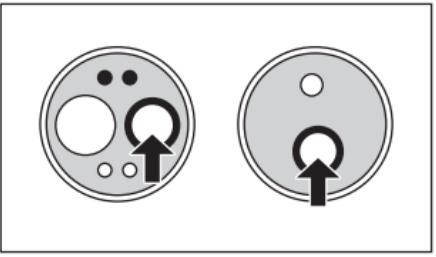
8

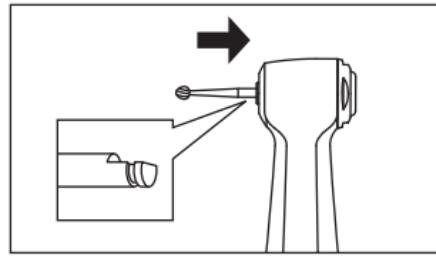
9

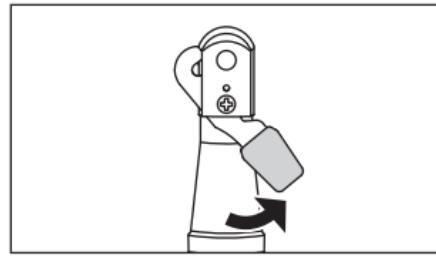
10

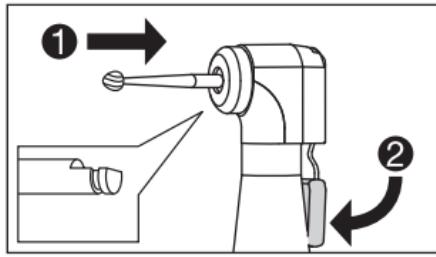
11

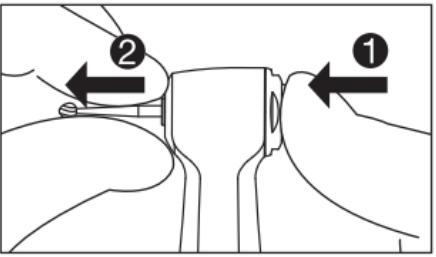
12

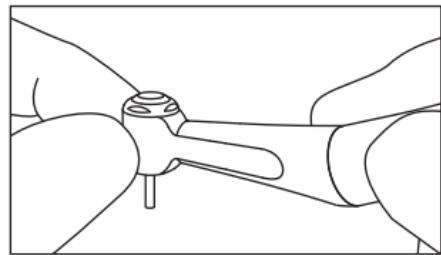
13

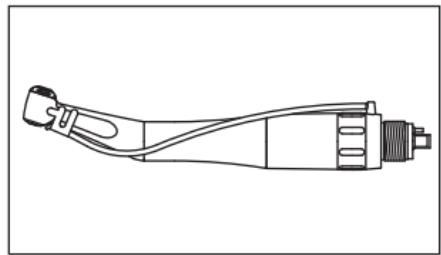
14

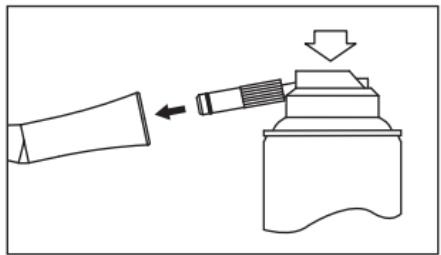
15

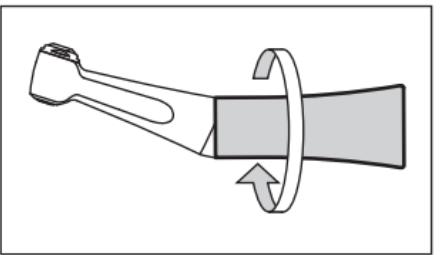
16

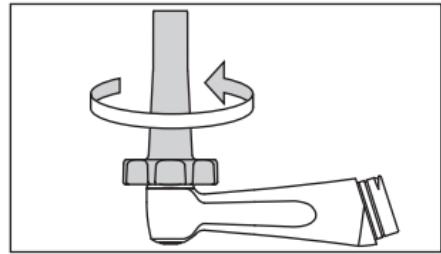
17

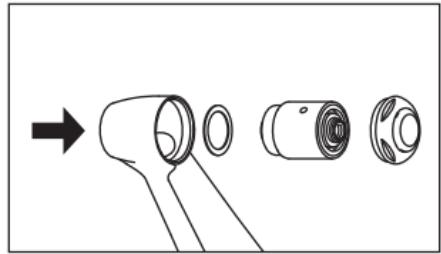
18

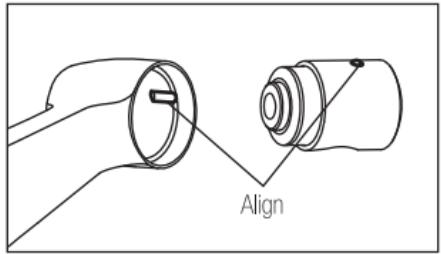
19

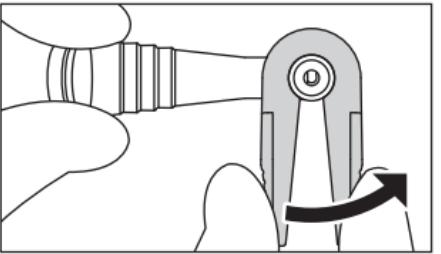
20

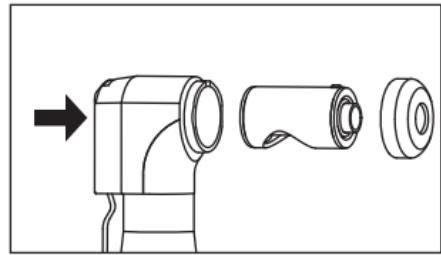21

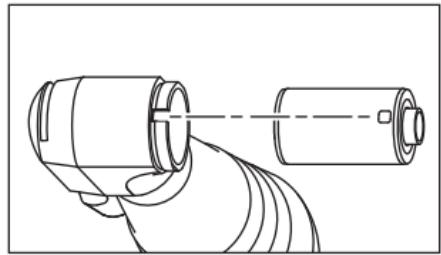22

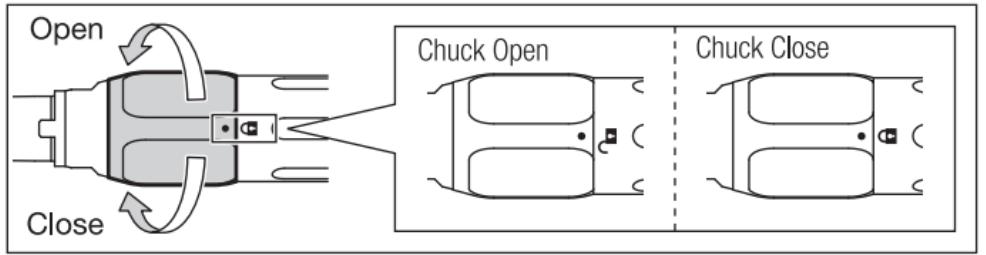23

## ◆ Set Component List

	Air motor				Contra Angle Handpiece				Straight Handpiece	
	FX205 M4	FX205 B2	FX205m M4	FX205m B2	FX22	FX23	FX25	FX25m	FX65	FX65m
FX205 M4 Set	○					○			○	
FX205 B2 Set		○				○			○	
FX205 M4/FX22 Set	○				○				○	
FX205 B2/FX22 Set		○			○				○	
FX205 M4/FX25 Set	○					○			○	
FX205 B2/FX25 Set		○				○			○	
FX205m M4 Set			○					○		○
FX205m B2 Set				○				○		○

## ◆ User and Indications for Use

User : Qualified Professionals

Indications for Use:

### FX205 M4/FX205 B2/FX205m M4/FX205m B2

This medical device is intended for converting compressed air supplied from a dental unit, etc. to rotational motion as well as transmitting the rotation to the instrument for dental treatment such as a straight or geared angle handpiece to cut and polish natural or artificial teeth.

### FX22/FX23/FX25/FX25m

This medical device is intended for transmitting rotation of the power source at the direct drive ratio, thereby running the instrument such as a bur to cut and polish natural or artificial teeth during dental treatment.

### FX65/FX65m

This medical device is intended for transmitting rotation of the power source at the direct drive ratio, thereby running the instrument such as a bur to cut and polish natural or artificial teeth during dental treatment.

## ◆ Accessory List

No.	Part Name	Quantity	Remarks
1	FX Wrench	1	FX205 M4 Set FX205 M4/FX22 Set FX205 M4/FX25 Set FX205m M4 Set
2	Cap Wrench	1	FX205 M4/FX22 Set
3	FX Spray Nozzle	1	FX205 M4 Set FX205 M4/FX25 Set FX205m M4 Set
4	Spray Nozzle	1	FX205 M4/FX22 Set
5	E-Type Spray Nozzle	1	-
6	Turbine Oil	1	-

## ◆ Precautions for handling and operation

- Please read these precautions carefully and use only as intended or instructed.
- Safety instructions are intended to avoid potential hazards that could result in personal injury or damage to the device. Safety instructions are classified as follows in accordance with the seriousness of the risk.

Class	Degree of Risk
<b>! WARNING</b>	Hazard that could result in serious injury or damage to the device if the safety instructions are not correctly followed.
<b>! CAUTION</b>	Hazard that could result in light or moderate injury or damage to the device if the safety instructions are not correctly followed.
<b>N O T I C E</b>	General product specification information highlighted to avoid product malfunction and performance reduction.

## Air Motor (FX205 M4/FX205m M4/FX205 B2/FX205m B2)

### ! CAUTION

- Read this Operation Manual before use to fully understand the product functions and file for future reference.
- When operating the product always consider the safety of the patient.
- Users are responsible for the operational control, maintenance and continual inspection of this product.
- Do not disassemble or alter the motor except as recommended by NSK in this Operation Manual.
- Do not allow any impact on to the product. Do not drop the product.
- Operators and all others in the area must wear eye protection and a mask when operating this motor.
- Should the motor function abnormally during use, stop using the motor immediately and contact your Authorized NSK Dealer.
- Do not use the following fluids to wipe, immerse or clean the product; strong/super acid water, strong acid/alkaline chemicals, chlorine-containing solutions, solvents such as benzine or thinner.
- The motor is delivered in a non-sterile condition and must be autoclaved prior to use.
- Perform regular function and maintenance checks. (Refer to "7. Periodical Maintenance Checks")
- If the motor has not been used for a long period, rotate the motor and check for noise, vibration and overheating before use.
- To avoid clinical downtime it is recommended that a spare be kept on hand in case of a breakdown during treatment.
- U.S. Federal law restricts this device to sale by or on the order of a licensed physician.

## 1. Setting of Air & Water Supply Pressure

Measure the supply pressure at the handpiece / hose connection point and set the pressure to the value specified on the specification table. (Fig. 1)

For Multi Gauge information refer to Option Parts List.

### ⚠ WARNING

- Do not exceed the optimum pressure specified on the specification table.

### ⚠ CAUTION

- Do not use air contaminated by dust, moisture and oil.

## 2. Connection & Disconnection of the Handpiece

### 2-1 Connection

- 1) Connect the handpiece direct to the motor. (Fig. 2)
- 2) Make sure the handpiece is firmly connected to the motor.

### 2-2 Disconnection

Hold the motor and the handpiece separately and disconnect with care.

### ⚠ CAUTION

- Disconnect the handpiece only after the motor has completely stopped rotating.

## 3. Check before treatment

Connect the handpiece to the motor and check for motor vibration, noise and overheating. If any abnormalities are found do not use the motor and contact your Authorized NSK Dealer.

## 4. Water and Chip Air Supply to the Handpiece

- 1) Attach the attached FX Spray nozzle to the neck of the Contra Angle handpiece.
- 2) Trim the tube to the desired length then insert the tube end into the nozzle clamp on the motor. (Fig. 3)

## 5. Forward / Reverse Rotation

- 1) Forward Rotation : Turn the FWD/REV Change Ring to "F". (Fig. 4)
- 2) Reverse Rotation : Turn the FWD/REV Change Ring to "R". (Fig. 5)
- 3) When the FWD/REV Change Ring is positioned at the middle of "F" and "R" , the drive air is interrupted and the motor does not rotate. (Fig. 6)  
F : Forward Rotation    R : Reverse Rotation

## 6. Post-use Maintenance

After each patient maintain the product as follows.

### ⚠ WARNING

- Immediately after a treatment, the motor should be cleaned, lubricated and sterilized. Failure to properly maintain the motor may lead to overheating, causing burn injuries or product failure.

### ⚠ CAUTION

- Do not use the following fluids to wipe, immerse or clean the product; strong/super acid water, strong acid/alkaline chemicals, chlorine-containing solutions, solvents such as benzene or thinner.

### 6-1 Preparation

- 1) Wear a eye protection, a mask and gloves to avoid infection.
- 2) Disconnect the handpiece from the motor.
- 3) Remove the motor from the hose.

## 6-2 Cleaning

### ■ Manual Cleaning (External Surface)

- 1) Remove dirt and debris on the motor surface using a brush under running water (at 38°C or less). Do not use a wire brush.
- 2) Wipe off the moisture on the surface using absorbent cloth.

### ■ Automatic Cleaning (External and Internal Surfaces) (Only for FX205m M4/FX205m B2)

Use a thermo-disinfector in accordance with ISO 15883 (EN ISO 15883).

Refer to the thermo-disinfector manual.

### ⚠ CAUTION

- After washing with thermo-disinfector and prior to lubrication, dry the product until all internal moisture is thoroughly removed. Thermo-disinfector moisture remaining inside the product could reduce the effect of lubrication and could cause corrosion inside of the product.

## 6-3 Disinfection

### ■ Manual Disinfection (External Surface)

Wipe the surface with cloth soaked in disinfectant solution such as alcohol.

### ■ Automatic Disinfection (External and Internal Surfaces) (Only for FX205m M4/FX205m B2)

Use a thermo-disinfector in accordance with ISO 15883 (EN ISO 15883).

Refer to the thermo-disinfector manual.

## 6-4 Lubrication

### ■ NSK PANA SPRAY Plus / PANA SPRAY

Apply NSK PANA SPRAY Plus / PANA SPRAY every time after each use and/or before autoclaving.

- 1) Remove the motor from the hose.
- 2) Mount the arrow-head spray nozzle tip into the spray can port. (Fig. 7)

- 3) Insert the Spray Nozzle into the Drive Air Port of the motor. Hold the motor and spray for approximately 2-3 seconds. Apply lubricant until it expels from the exhaust air part for at least 2 seconds. (Fig. 8)

### ⚠ CAUTION

- Hold the spray can upright.
- Be sure to hold the motor firmly to prevent slipping out of hand by the spray pressure when lubricating.

### NOTICE

- NSK recommends the use of "Spray Mist Absorber" (Y900084) to prevent oil mist expelling out of the motor.

### ■ Lubrication Oil

- 1) Remove the motor from the hose.
- 2) Apply 1-2 drops of lubrication oil into the Drive Air Port shown in the Fig. 8.
- 3) Connect the motor to the hose. Operate the motor for about 10 seconds.

## 6-5 Sterilization

Sterilize the product by autoclave sterilization. After each patient sterilize as noted below.

- 1) Insert into an autoclave pouch. Seal the pouch.
- 2) Autoclavable under the conditions below.

Autoclave for more than 20min. at 121°C, or 15min. at 132°C, or 3min. at 134°C.

- 3) The motor should remain in the autoclave pouch until required for use.

**CAUTION**

- Do not autoclave the product with other instruments even when it is in a pouch. This is to prevent possible discoloration and damage to the product from chemical residue on other instruments.
- Do not heat or cool the product too quickly. Rapid change in temperature could cause damage to the product.
- To avoid product failure, do not use a sterilizer whose temperature exceeds 138°C during use including the dry cycle. The chamber temperature of some type of sterilizers may exceed 138°C. Contact the sterilizer manufacturer for detailed information.
- Keep the product in suitable atmospheric pressure, temperature, humidity, ventilation, and sunlight. The air should be free from dust, salt and sulphur.
- Do not touch the product immediately after autoclaving as it will be very hot and must remain in a sterile condition.
- Autoclave sterilization is recommended for the product. The validity of other sterilization methods is not confirmed.

**NOTICE**

- NSK recommends Class B sterilizers as stated in EN13060.

**7. Periodical Maintenance Checks**

Perform periodical maintenance checks every three months, referring to the check sheet below. If any abnormalities are found, contact your Authorized NSK Dealer.

Points to check	Details
Rotation	Rotate the motor and check for abnormalities such as abnormal rotation, vibration, noise, and overheating.

**8. Specifications**

Model	FX205 M4	FX205m M4	FX205 B2	FX205m B2
Hose Connection Type	ISO 9168 Type 3 (Midwest 4 hole)	ISO 9168 Type 1 (Borden 2 hole)		
Handpiece Connection Type		E Type ISO 3964		
Max. Rotation Speed	19,800-24,200min <sup>-1</sup> (0.25MPa)	22,500-27,500min <sup>-1</sup> (0.25MPa)		
Breakdown Torque		1.5N·cm and more(0.25MPa)		
Drive Air Pressure		0.20 - 0.25MPa (2.0 - 2.5kgf/cm <sup>2</sup> )		
Air Consumption		<66NL/min(0.25MPa)		
Water Pressure		0.10 - 0.25MPa (1.0 - 2.5kgf/cm <sup>2</sup> )		
Chip Air Pressure	0.15 - 0.25MPa (1.5 - 2.5kgf/cm <sup>2</sup> )		-	
Coolant Water Supply		50mL/min and more (0.20MPa)		
Coolant Air Supply	1.5NL/min and more (0.20MPa)		-	
Water Spray Type		External Spray		
Use Environment		Temperature: 10 - 40°C, Humidity: 30 - 75% (No Condensation)		
Transportation and Store Environment		Temperature: -10 - 50°C, Humidity: 10 - 85%, Atmospheric Pressure: 500 - 1,060hPa		
Thermo Disinfecter	-	O	-	O

## Contra Angle Handpiece (FX22/FX23/FX25/FX25m)

### ⚠️ WARNING

- Immediately after a treatment, the handpiece should be cleaned, lubricated and sterilized. Failure to properly maintain the handpiece may lead to overheating, causing burn injuries or product failure. (Refer to "13. Post-use Maintenance")
- If blood infiltrates inside a handpiece, an automatic handpiece cleaning and lubrication system may not totally clean the internal handpiece components, which may lead to internal blood coagulation. Coagulated blood inside the handpiece may cause handpiece failure and overheating, causing burn injuries. To avoid the risk, NSK recommends to immediately lubricate those handpieces with PANA SPRAY Plus / PANA SPRAY.
- Depressing the Push Button while the handpiece is in rotation may lead to overheating, causing burn injuries or product failure. Avoid the push button to contact with any oral tissue. (Except for FX22)
- Supply coolant water and coolant air while using the handpiece. No supplying the coolant water and coolant air may lead to overheating, causing burn injuries or product failure.
- Do not use the handpiece for polishing purposes. Polishing paste could enter the handpiece, causing the failure of the Push Button or burn injuries due to the overheating of the handpiece head.
- Keep any debris or other foreign materials away from inside the gear or the handpiece. Foreign materials remaining inside may lead to overheating, causing burn injuries or other accidents.
- Ball bearings are wear items and may require replacement. Be sure to inspect the handpiece before use (Refer to "11. Check before treatment"). If abnormal vibration or noise are found, the bearings or other internal parts may be worn out or damaged. Damaged parts may lead to handpiece overheating causing burn injuries. If overheating is suspected, stop using the handpiece immediately and contact your Authorized NSK Dealer.

### ⚠️ CAUTION

- When operating the product always consider the safety of the patient.
- Users are responsible for the operational control, maintenance and continual inspection of this product.
- Prior to clinical use, inspect the handpiece. Check for vibration, noise and overheating. If any abnormalities are found, stop using the handpiece immediately and contact your Authorized NSK Dealer. (Refer to "11. Check before treatment")
- Do not disassemble or alter the handpiece except as recommended by NSK in this Operation Manual.
- Do not allow any impact on to the product. Do not drop the product.
- Operators and all others in the area must wear eye protection and a mask when operating this handpiece.
- Do not use burs with problems listed below as the bur may break, seize up or disengage from the chuck.
  - Bent, deformed, worn, rusted, broken, deficient bur.
  - Bur which is cracked on the edge or axis.
  - Non-ISO(EN ISO) standard, or tampered bur.
- Do not use burs other than specified in "16. Specifications" as such burs may be accidentally released or break during rotation.
- Always keep the bur shank clean. Dirt or debris in the chuck could cause poor bur concentricity or low chuck retention force.
- Always insert the bur all the way into the chuck. If insertion is insufficient, premature failure of the bearings or accidental release of the bur may occur.
- Do not exceed Max. Bur Length shown in "16. Specifications".
- Always follow the instructions provided by the bur manufacturer.
- Do not exceed the bur speed recommended by the bur manufacturer.
- Should the handpiece function abnormally during use, stop using the handpiece immediately and contact your Authorized NSK Dealer.
- Do not use the following fluids to wipe, immerse or clean the product; strong/super acid water,

- strong acid/alkaline chemicals, chlorine-containing solutions, solvents such as benzine or thinner.
- The handpiece is delivered in a non-sterile condition and must be autoclaved prior to use.
  - Perform regular function and maintenance checks. (Refer to "15. Periodical Maintenance Checks")
  - If the handpiece has not been used for a long period, rotate the handpiece and check for noise, vibration and overheating before use.
  - To avoid clinical downtime it is recommended that a spare be kept on hand in case of a breakdown during treatment.
  - The operation of the handpiece is permitted only on dental units which correspond to the standards IEC 60601-1 (EN 60601-1) and IEC 60601-1-2 (EN 60601-1-2).
  - U.S. Federal law restricts this device to sale by or on the order of a licensed physician.

## 9. Connection & Disconnection of the Handpiece and Motor

### 9-1 Connection

- 1) Connect the handpiece direct to the motor. (Fig. 2)
- 2) Make sure the handpiece is firmly connected to the motor.

### 9-2 Disconnection

Hold the motor and the handpiece separately and disconnect with care.

#### ⚠ CAUTION

- Do not connect or disconnect the handpiece until the motor has completely stopped.
- Connect ONLY to E type motors (ISO 3964 (EN ISO 3964)).
- Do not exceed the Max. Rotation Speed (Motor) shown in "16. Specifications".

## 10. Insertion & Removal of the Bur

### 10-1 To Insert the Bur

[FX23/FX25/FX25m]

- 1) Insert the bur until it is correctly seated in place.
- 2) Depress the Push Button and insert the bur into the chuck until the bur "notch" mechanism engages. Release the button. (Fig. 9)
- 3) Ensure that the bur is secure by gently pulling and pushing the bur WITHOUT depressing the Push Button.

[FX22]

- 1) Turn the Latch Plate to open the chuck. (Fig. 10)
- 2) Insert the bur into the chuck until the bur securely engages. Swing the Latch Plate back to closed position. (Fig. 11)
- 3) Ensure that the bur is secure by gently pulling and pushing the bur.

### 10-2 To Remove the Bur

[FX23/FX25/FX25m]

Depress the Push Button to open the chuck (①) and remove the bur (②). (Fig. 12)

[FX22]

Turn the Latch Plate to open position and pull the bur out.

#### ⚠ CAUTION

- Do not use burs with problems listed below as the bur may break, seize up or disengage from the chuck.
  - Bent, deformed, worn, rusted, broken, deficient bur.
  - Bur which is cracked on the edge or axis.
  - Non-ISO(EN ISO) standard, or tampered bur.

- Do not use burs other than specified in "16. Specifications" as such burs may be accidentally released or break during rotation.
- Always keep the bur shank clean. Dirt or debris in the chuck could cause poor bur concentricity or low chuck retention force.
- Always insert the bur all the way into the chuck. If insertion is insufficient, premature failure of the bearings or accidental release of the bur may occur.
- Do not exceed Max. Bur Length shown in "16. Specifications".
- Always follow the instructions provided by the bur manufacturer.
- Do not exceed the bur speed recommended by the bur manufacturer.
- Do not insert or remove the bur until the rotation has completely stopped.
- Do not apply excess pressure to the bur as it may break or bend or become difficult to remove.

## 11. Check before treatment

Follow the check procedure below before use. If any abnormalities are found, stop using the handpiece immediately and contact your Authorized NSK Dealer.

- 1) Check the Head Cap or Cap (FX22) is firmly tightened.
- 2) Insert the bur. (Refer to "10. Insertion & Removal of the Bur")
- 3) Rotate the handpiece for about one minute at the Max. rotation speed of the attached motor. During rotation, check for abnormalities such as abnormal rotation, vibration, noise.
- 4) After the handpiece rotation has completely stopped, touch the handpiece head to confirm the head is NOT heating abnormally (Fig. 13).

### ⚠ CAUTION

- To avoid injury, keep your hands away from the bur during rotation.

## 12. Connection of the Spray Nozzle

- 1) Attach the Spray Nozzle to the handpiece head (Fig. 14).
- 2) Cut the Irrigation Tube to the desired length then connect to the Spray Tube Nozzle on the motor (Fig. 14).

## 13. Post-use Maintenance

After each patient maintain the product as follows.

### ⚠ WARNING

- Immediately after a treatment, the handpiece should be cleaned, lubricated and sterilized. Failure to properly maintain the handpiece may lead to overheating, causing burn injuries or product failure.

### ⚠ CAUTION

- Do not use the following fluids to wipe, immerse or clean the product; strong/super acid water, strong acid/alkaline chemicals, chlorine-containing solutions, solvents such as benzine or thinner.

### 13-1 Preparation

- 1) Wear a eye protection, a mask and gloves to avoid infection.
- 2) Remove the bur.
- 3) Disconnect the handpiece from the motor.

### 13-2 Cleaning

#### ■ Manual Cleaning (External Surface)

- 1) Remove dirt and debris on the handpiece surface using a brush under running water (at 38°C or less).  
Do not use a wire brush.
- 2) Wipe off the moisture on the surface using absorbent cloth.

## CAUTION

- If water comes inside the handpiece, blow air into the rear of the handpiece using an air syringe and remove the remaining water.

### ■ Automatic Cleaning (External and Internal Surfaces) (Only for FX25m)

Use a thermo-disinfector in accordance with ISO 15883 (EN ISO 15883). Refer to the thermo-disinfector manual.

## CAUTION

- After washing with thermo-disinfector and prior to lubrication, dry the product until all internal moisture is thoroughly removed. Thermo-disinfector moisture remaining inside the product could reduce the effect of lubrication and could cause corrosion inside of the product.

### 13-3 Disinfection

#### ■ Manual Disinfection (External Surface)

Wipe the surface with cloth soaked in disinfectant solution such as alcohol.

#### ■ Automatic Disinfection (External and Internal Surfaces) (Only for FX25m)

Use a thermo-disinfector in accordance with ISO 15883 (EN ISO 15883). Refer to the thermo-disinfector manual.

### 13-4 Lubrication

#### ■ NSK PANA SPRAY Plus / PANA SPRAY



Apply NSK PANA SPRAY Plus / PANA SPRAY every time after each use and/or before autoclaving.

1) Attach the Spray Nozzle into the Spray Port nozzle on the can.

2) Insert the Spray Nozzle into the rear of the handpiece. Hold the handpiece and spray for approximately 2-3 seconds. Apply lubricant until it expels from the handpiece head for at least 2 seconds (Fig. 15).

## CAUTION

- Hold the spray can upright.
- Be sure to hold the handpiece firmly to prevent slipping out of hand by the spray pressure when lubricating.
- Supply lubricant until it comes out of the handpiece head.
- If you wish to expel excessive oil from inside of the handpiece, rotate the handpiece for approx.15 seconds without a bur. During the rotation, do not depress the push button inadvertently, especially when using absorbent cloth to prevent oil from scattering. Depressing the push button during rotation may cause a failure to attach a bur. (Except for FX22)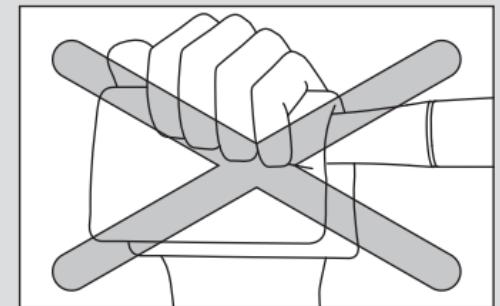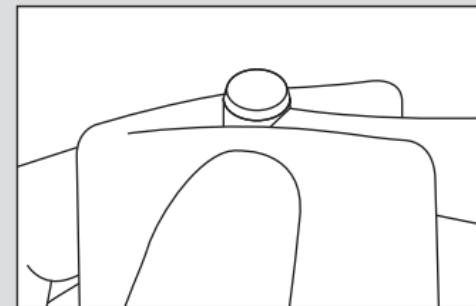

## NOTICE

- NSK recommends the use of "Spray Mist Absorber" (Y900084) to prevent oil mist expelling out of the handpiece head.

## ■ NSK automatic handpiece cleaning and lubrication system

When using NSK automatic handpiece cleaning and lubrication system, refer to the system instructions.

### ⚠️ WARNING

- If blood infiltrates inside a handpiece, an automatic handpiece cleaning and lubrication system may not totally clean the internal handpiece components, which may lead to internal blood coagulation. Coagulated blood inside the handpiece may cause handpiece failure and overheating, causing burn injuries. To avoid the risk, NSK recommends to immediately lubricate those handpieces with PANA SPRAY Plus / PANA SPRAY.

### 13-5 Sterilization

Sterilize the product by autoclave sterilization. After each patient sterilize as noted below.

1) Insert into an autoclave pouch. Seal the pouch.

2) Autoclavable under the conditions below.

    Autoclave for more than 20min. at 121°C, or 15min. at 132°C, or 3min. at 134°C.

3) The handpiece should remain in the autoclave pouch until required for use.

### ⚠️ CAUTION

- Be sure to clean and lubricate the handpiece prior to autoclaving. If blood remains on the internal surface it can become clotted and cause product failure.
- Do not autoclave the product with other instruments even when it is in a pouch. This is to prevent possible discoloration and damage to the product from chemical residue on other instruments.
- Do not heat or cool the product too quickly. Rapid change in temperature could cause damage to the product.
- To avoid product failure, do not use a sterilizer whose temperature exceeds 138°C during use including the dry cycle. The chamber temperature of some type of sterilizers may exceed 138°C. Contact the sterilizer manufacturer for detailed information.
- Keep the product in suitable atmospheric pressure, temperature, humidity, ventilation, and sunlight. The air should be free from dust, salt and sulphur.

- Do not touch the product immediately after autoclaving as it will be very hot and must remain in a sterile condition.

- Autoclave sterilization is recommended for the product. The validity of other sterilization methods is not confirmed.

### NOTICE

- NSK recommends Class B sterilizers as stated in EN 13060.

## 14. Replacing the Cartridge


1) Detach the shank by turning it counter clockwise and remove the head from the shank (Fig.16).

### ⚠️ CAUTION

- After removing, do not hold the shank upside down as it could cause the internal parts to drop.

2) Insert the wrench firmly into the rear of the handpiece head until it stops, as shown in Fig. A-1.

3) Tightly hold the FX wrench as illustrated in Fig. B then pull the drive shaft straight out in the direction of the arrow (Fig. B).

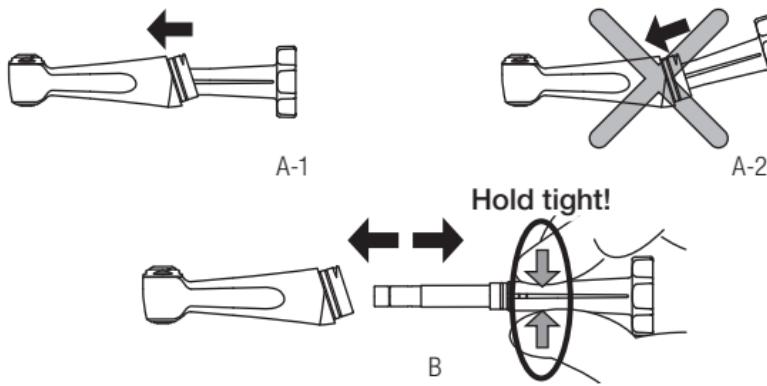### CAUTION

- Be sure to insert the FX wrench straight into the handpiece head. Do not insert the FX wrench at an angle as shown in Fig. A-2.
- When removing the drive shaft, tightly hold the FX wrench as directed in Fig. B.

### FX23/FX25/FX25m

- 4) Locate the correct wrench tool for the head cap then turn the wrench counter clockwise to loosen the cap. Remove the cap (Fig. 17).
- 5) Use the bur to gently push the entire cartridge out from the head (Fig. 18).
- 6) Clean the head interior with NSK PANA SPRAY Plus / PANA SPRAY.
- 7) Wipe the NSK PANA SPRAY Plus / PANA SPRAY oil over the head interior.
- 8) Insert the new cartridge into the head by aligning the pin on the cartridge with the slot on the head (Fig. 19).
- 9) Tighten the head cap with the correct wrench.
- 10) Insert the drive shaft firmly into the handpiece head until it clicks. If the click is not heard, insert it again.

again.

11) Reattach the shank.

\*Refer to "25. Spare Parts List" to identify the correct cartridge.

### FX22

- 4) Locate the correct Cap Wrench tool for the cap then turn the cap wrench counter clockwise to loosen the cap. Remove the cap (Fig. 20).
  - 5) Use the bur to gently push the entire cartridge out from the head (Fig. 21).
  - 6) Clean the head interior with NSK PANA SPRAY Plus / PANA SPRAY.
  - 7) Wipe the NSK PANA SPRAY Plus / PANA SPRAY oil over the head interior.
  - 8) Insert the new cartridge into the head by aligning the pin on the cartridge with the slot on the head (Fig. 22).
  - 9) Tighten the cap with the correct cap wrench.
  - 10) Insert the drive shaft firmly into the handpiece head until it clicks. If the click is not heard, insert it again.
  - 11) Reattach the shank.
- \*Refer to "25. Spare Parts List" to identify the correct cartridge.

### CAUTION

- Before turning the head cap, make sure the wrench is correctly aligned with the cap. Do not turn the wrench at an angle as abrasion of the wrench may occur.
- Secure the shank firmly to the handpiece head so that there is no gap between them. A loose connection could make the head unstable, resulting in an accident during use.
- Use only a genuine NSK cartridge. If a non genuine cartridge is used NSK cannot guarantee performance and the handpiece warranty would become invalid.
- Always first finger tighten the head cap then secure firmly with the wrench.
- NSK never recommends the disassembly and repair of any NSK cartridge. There is NO EXCEPTION. In such a case a handpiece may perform abnormally (abnormal noise or abnormal vibration). Damage, failure or accidents are outside of our guarantee.

## 15. Periodical Maintenance Checks

Every 3 months perform periodical maintenance checks, referring to the check sheet below. If any abnormalities are found, contact your Authorized NSK Dealer.

Points to check	Details
Head cap or Cap (FX22) is loose	Tighten firmly using the correct head cap wrench or cap wrench.
Rotation	Rotate the handpiece and check for abnormalities such as abnormal rotation, vibration, noise, and overheating.

## 16. Specifications

Model	FX22	FX23	FX25	FX25m			
Max. Rotation Speed (Motor)	30,000 min <sup>-1</sup>		40,000 min <sup>-1</sup>				
Max. Rotation Speed (Handpiece)	30,000 min <sup>-1</sup>		40,000 min <sup>-1</sup>				
Gear Ratio	1:1 Direct Drive						
Bur Type	ISO 1797-1 Type1 ø2.35mm CA Bur						
Chuck Type	Latch Type	Push Button Type					
Chucking Length	13.2mm	12.7mm					
Max. Bur Length		22.5mm					
Max. Working Part Diameter		ø4.0mm					
Use Environment	Temperature: 10 - 40°C, Humidity: 30 - 75% (No Condensation)						
Transportation and Store Environment	Temperature: -10 - 50°C, Humidity: 10 - 85%, Atmospheric Pressure: 500 - 1,060hPa						
Thermo Disinfector	-		○				

## Straight Handpiece (FX65/FX65m)

### !CAUTION

- Read this Operation Manual before use to fully understand the product functions and file for future reference.
- When operating the product always consider the safety of the patient.
- Users are responsible for the operational control, maintenance and continual inspection of this product.
- Do not disassemble or alter the handpiece except as recommended by NSK in this Operation Manual.
- Do not allow any impact on to the product. Do not drop the product.
- Operators and all others in the area must wear eye protection and a mask when operating this handpiece.
- Should the handpiece function abnormally during use, stop using the handpiece immediately and contact your Authorized NSK Dealer.
- Do not use the following fluids to wipe, immerse or clean the product; strong/super acid water, strong acid/alkaline chemicals, chlorine-containing solutions, solvents such as benzine or thinner.
- The handpiece is delivered in a non-sterile condition and must be autoclaved prior to use.
- Perform regular function and maintenance checks. (Refer to "21. Periodical Maintenance Checks")
- If the handpiece has not been used for a long period, rotate the handpiece and check for noise, vibration and overheating before use.
- To avoid clinical downtime it is recommended that a spare be kept on hand in case of a breakdown during treatment.
- U.S. Federal law restricts this device to sale by or on the order of a licensed physician.

## 17. Connection & Disconnection of the Handpiece and Motor

### 17-1 Connection

- 1) Connect the handpiece direct to the motor. (Fig. 2)
- 2) Make sure the handpiece is firmly connected to the motor.

### 17-2 Disconnection

Hold the motor and the handpiece separately and disconnect with care.

#### CAUTION

- Disconnect the handpiece only after the motor has completely stopped rotating.

## 18. Insertion & Removal of the Bur

### 18-1 To Insert the Bur

- 1) Turn the Bur Lock Ring in the direction of the "OPEN" mark until it clicks. (Fig. 23)
- 2) Insert the bur fully into the chuck then turn the Bur Lock Ring in the direction of the mark "LOCK" until it clicks.
- 3) Test that the bur is secure by gently pulling and pushing the bur.

### 18-2 To Remove the Bur

Turn the Bur Lock Ring in the direction of the "OPEN" mark until it clicks then remove the bur.

#### WARNING

- Make sure to align the • mark with "LOCK" mark. If those marks are not completely aligned, it may cause handpiece overheating or patient burn injuries.

#### CAUTION

- Always insert the bur fully into the chuck. Insufficient insertion of the bur could cause premature failure of the bearing.
- Remove the bur only after the handpiece has completely stopped rotating.
- Do not rotate the handpiece when the Bur Lock Ring is turned to "OPEN" (OPEN) position or a bur is not inserted.
- Always keep the bur shank clean. Entry of debris into the chuck, via the bur shank, could cause bur rotation slip and also prevent the bur from being securely located in the chuck.
- Do not exceed the bur speed recommended by the bur manufacturer.
- Do not exceed maximum bur length recommended by the handpiece manufacturer.
- Do not apply excess pressure to the bur as it may break or bend or become difficult to remove.
- DO NOT use burs with problems listed below as the bur may break or disengage from the chuck.
  - Bent, deformed, anisomeric (worn), rusted, broken, deficient bur.
  - Bur which is cracked on the edge or axis.
  - Non-ISO standard, or tampered bur.

## 19. Check before treatment

Check for handpiece vibration, noise and overheating. If any abnormalities are found do not use the handpiece and contact your Authorized NSK Dealer.

#### CAUTION

- To avoid injury, keep your hands away from the bur during rotation.

## 20. Post-use Maintenance

After each patient maintain the product as follows.

### ⚠ WARNING

- Immediately after a treatment, the handpiece should be cleaned, lubricated and sterilized. Failure to properly maintain the handpiece may lead to overheating, causing burn injuries or product failure.

### ⚠ CAUTION

- Do not use the following fluids to wipe, immerse or clean the product; strong/super acid water, strong acid/alkaline chemicals, chlorine-containing solutions, solvents such as benzene or thinner.

### 20-1 Preparation

- 1) Wear a eye protection, a mask and gloves to avoid infection.
- 2) Remove the bur.
- 3) Disconnect the handpiece from the motor.

### 20-2 Cleaning

#### ■ Manual Cleaning (External Surface)

- 1) Remove dirt and debris on the handpiece surface using a brush under running water (at 38°C or less).  
Do not use a wire brush.
- 2) Wipe off the moisture on the surface using absorbent cloth.

### ⚠ CAUTION

- If water comes inside the handpiece, blow air into the rear of the handpiece using an air syringe and remove the remaining water.

■ Automatic Cleaning (External and Internal Surfaces) (Only for FX65m)  
Use a thermo-disinfector in accordance with ISO 15883 (EN ISO 15883). Refer to the thermo-disinfector manual.

### ⚠ CAUTION

- After washing with thermo-disinfector and prior to lubrication, dry the product until all internal moisture is thoroughly removed. Thermo-disinfector moisture remaining inside the product could reduce the effect of lubrication and could cause corrosion inside of the product.

### 20-3 Disinfection

#### ■ Manual Disinfection (External Surface)

Wipe the surface with cloth soaked in disinfectant solution such as alcohol.

#### ■ Automatic Disinfection (External and Internal Surfaces) (Only for FX65m)

Use a thermo-disinfector in accordance with ISO 15883 (EN ISO 15883). Refer to the thermo-disinfector manual.

### 20-4 Lubrication

#### ■ NSK PANA SPRAY Plus / PANA SPRAY

Apply NSK PANA SPRAY Plus / PANA SPRAY every time after each use and/or before autoclaving.

- 1) Attach the Spray Nozzle into the Spray Port nozzle on the can.
- 2) Insert the Spray Nozzle into the rear of the handpiece. Hold the handpiece and spray for approximately 2-3 seconds. Apply lubricant until it expels from the handpiece head for at least 2 seconds (Fig. 15).

**CAUTION**

- Hold the spray can upright.
- Be sure to hold the handpiece firmly to prevent slipping out of hand by the spray pressure when lubricating.
- Supply lubricant until it comes out of the handpiece head.

**NOTICE**

- NSK recommends the use of "Spray Mist Absorber" (Y900084) to prevent oil mist expelling out of the handpiece head.

**NSK automatic handpiece cleaning and lubrication system**

When using NSK automatic handpiece cleaning and lubrication system, refer to the system instructions.

**WARNING**

- If blood infiltrates inside a handpiece, an automatic handpiece cleaning and lubrication system may not totally clean the internal handpiece components, which may lead to internal blood coagulation. Coagulated blood inside the handpiece may cause handpiece failure and overheating, causing burn injuries. To avoid the risk, NSK recommends to immediately lubricate those handpieces with PANA SPRAY Plus / PANA SPRAY.

**20-5 Sterilization**

Sterilize the product by autoclave sterilization. After each patient sterilize as noted below.

- 1) Insert into an autoclave pouch. Seal the pouch.
- 2) Autoclavable under the conditions below.  
Autoclave for more than 20min. at 121°C, or 15min. at 132°C, or 3min. at 134°C.
- 3) The handpiece should remain in the autoclave pouch until required for use.

**CAUTION****CAUTION**

- Be sure to clean and lubricate the handpiece prior to autoclaving. If blood remains on the internal surface it can become clotted and cause product failure.
- Do not autoclave the product with other instruments even when it is in a pouch. This is to prevent possible discoloration and damage to the product from chemical residue on other instruments.
- Do not heat or cool the product too quickly. Rapid change in temperature could cause damage to the product.
- To avoid product failure, do not use a sterilizer whose temperature exceeds 138°C during use including the dry cycle. The chamber temperature of some type of sterilizers may exceed 138°C. Contact the sterilizer manufacturer for detailed information.
- Keep the product in suitable atmospheric pressure, temperature, humidity, ventilation, and sunlight. The air should be free from dust, salt and sulphur.
- Do not touch the product immediately after autoclaving as it will be very hot and must remain in a sterile condition.
- Autoclave sterilization is recommended for the product. The validity of other sterilization methods is not confirmed.

**NOTICE**

- NSK recommends Class B sterilizers as stated in EN 13060.

**21. Periodical Maintenance Checks**

Perform periodical maintenance checks every three months, referring to the check sheet below. If any abnormalities are found, contact your Authorized NSK Dealer.

Points to check	Details
Rotation	Rotate the handpiece and check for abnormalities such as abnormal rotation, vibration, noise, and overheating.

## 22. Specifications

Model	FX65	FX65m
Max. Rotation Speed (Motor)	40,000 min <sup>-1</sup>	
Max. Rotation Speed (Handpiece)	40,000 min <sup>-1</sup>	
Gear Ratio	1:1 Direct Drive	
Bur Type	ISO 1797-1 Type2 ø2.35mm HP Bur / ISO 14457 2012 Prophy Handpiece	
Chucking Length	30mm	
Max. Bur Length	44.5mm	
Max. Working Part Diameter	ø4mm	
Use Environment	Temperature: 10 - 40°C, Humidity: 30 - 75% (No Condensation)	
Transportation and Store Environment	Temperature: -10 - 50°C, Humidity: 10 - 85%, Atmospheric Pressure: 500 - 1,060hPa	
Thermo Disinfector	-	○

## Air Motor/Contra Angle Handpiece/Straight Handpiece

### 23. Symbol

 This product is Autoclavable up to Max.135°C.

 This product can be washed via Thermo Disinfector.

 Conforms to CE European Directive of "Medical device directive 93/42/EEC."

 Manufacturer.

 Authorized representative in the European community.

 Caution: U.S. Federal law restricts this device to sale by or on the order of a licensed physician.

### 24. Warranty

NSK products are warranted against manufacturing errors and defects in materials. NSK reserves the right to analyze and determine the cause of any problem. Warranty is voided should the product be not used correctly or for the intended purpose or has been tampered with by unqualified personnel or has had non NSK parts installed. Replacement parts are available for seven years beyond discontinuation of the model.

### 25. Option Parts List

Model	Order Code	Compatible Product
MG-4H Multi Gauge	Z109400	FX205 M4/FX205m M4
MG-2/3H Multi Gauge	Z109200	FX205 B2/FX205m B2
Spray Nozzle	H019711	FX65/FX65m

## 26. Spare Parts List

Model	Order Code	Compatible Product	Remarks
Spray Nozzle	C032701	FX22	-
FX Spray Nozzle	C1052702	FX23/FX25/FX25m	-
NAC-03	C030011	FX22	Cartridge
FX23-03	C1056700	FX23	Cartridge
FX25-03	C1052700	FX25/FX25m	Cartridge
FX25-04	C1052500	FX23/FX25/FX25m	Head Cap
FX22-02	10000996	FX22	Drive Shaft
FX25-02	10000946	FX23/FX25/FX25m	Drive Shaft
FX Wrench	C1052701	FX22/FX23/FX25/FX25m	-
Cap Wrench	C071751	FX22	-
E-Type Spray Nozzle	Z019090	FX22/FX23/FX25/FX25m/FX65/FX65m	-

## 27. Disposing product

In order to avoid the health risks of operators handling the disposal of medical equipment, as well as the risks of environmental contamination caused thereof, a surgeon or a dentist is required to confirm the equipment is sterile. Ask specialist firms who are licensed to dispose of specially controlled industrial wastes, to dispose the product for you.

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**NSK Europe GmbH** ■ [EC REP](#)